


instructions for use

Urine collection and storage using the **Mitra® Cartridge**

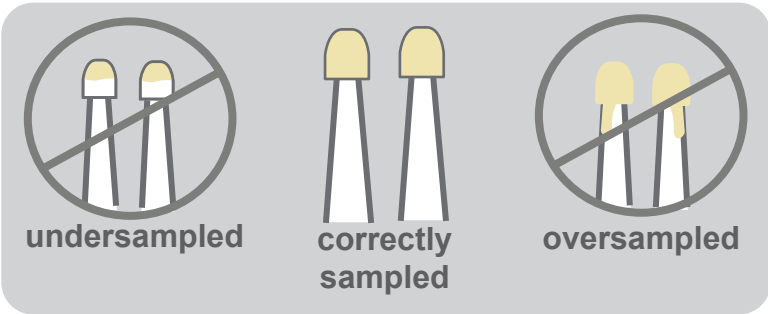


1 Prepare! See right-hand side.

2 Collect urine in cup mid-stream. Fill to almost full.

3 Replace lid on cup. Wash hands.

4 Wait ~5 minutes for urine bubbles to burst. Remove lid.

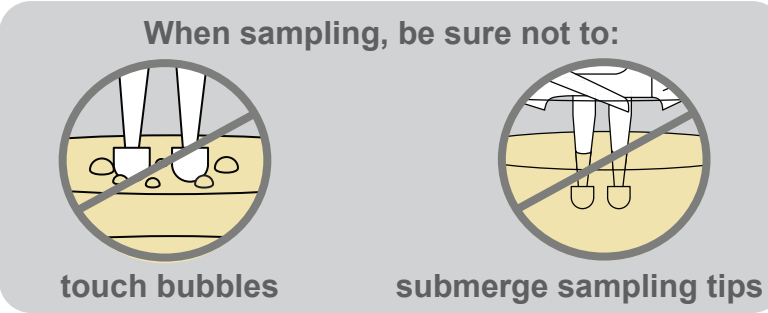

5 Touch both sampling tips to surface of urine at same time. Watch them become fully saturated, count 2 seconds, and SLOWLY remove.

6 Close Cartridge.

7 Insert Cartridge into specimen bag, seal bag shut, and insert in shipping envelope.

8 Discard urine and cup safely.

Do this FIRST!

Remove Cartridge from specimen bag

Do NOT remove desiccant.

Fill out the two labels with the same information

Affix one label to specimen bag and one to Cartridge as shown

Do NOT cover vents nor barcode

Open the Mitra® cartridge by pulling apart the vented flaps

Do NOT remove sampling tips from Cartridge

Remove lid from provided urine cup