JP

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Harpera™

Microbiopsy™ Punch(使い捨て)

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Microbiopsy™ Punch (Disposable)

Intended use

The Harpera™ Microbiopsy™ Punch for investigational use only (IUO) is intended to enable the collection of a specimen from the cutaneous skin surface by a healthcare professional for clinical studies.

Product description

The Harpera device is a single-use manual microbiopsy punch, a hand-held disposable device intended to achieve a controlled skin puncture in order to obtain a specimen. The punch is supplied individually packaged and sterile.

Instructions for use

Read IFU and watch IFU video prior to use the device.

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Neoteryx® and Harpera™ are trademarks owned by Trajan Scientific Australia Pty Ltd

Information and support	
	Trajan Scientific Australia Pty Ltd 7 Argent Place, Ringwood, VIC 3134, AU. Tel: +61 (0) 3 9874 8577
	Trajan Scientific Americas Inc 1421 W Wells Branch Parkway, Suite 108, Austin TX 78660, USA Tel: +1 512 837 7190
	Made in Australia
	www.neoteryx.com/collect-microbiopsy

Symbol meanings			
	Consult instructions for use or consult electronic instructions for use		Do not use if package is damaged and consult instructions for use
	Keep away from sunlight		Do not re-use
	Manufacturing date		Do not re-sterilize
	Keep dry		Caution
	Max. temp. 30°C		

For Investigational Use Only. The performance characteristics of this device have not been fully validated.

- Warnings and precautions

 - During activation, step 4-5, use with one hand.
 - Do not re-sterilize.
 - Do not re-use (single use).
 - Reuse of this device presents a risk of infection to the patient.
 - Handle with care so as not to injure yourself by touching the sharp tips/edges of the microbiopsy collector.
 - If the tip of the microbiopsy collector comes in contact with a foreign object, replace the device with a new one.
 - Do not use the device if any damage is observed to the packaging or the device itself.
 - If an incident occurs with the device or failures occur in collecting specimen, please contact neo.support@trajanscimed.com as soon as possible.

Storage and handling
In its sterile packaging, the device can be stored at room temperature. After use, dispose of the device properly in the biohazard waste. Dispose of the microbiopsy collector as sharp medical waste to prevent infection as per local guidance.

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