
Mitra[®] Device

Our Mitra[®] devices based on patented VAMS[®] technology provide an easy way to reliably collect a small, precise volume of blood. Collect microsamples in the lab, at home, or in the field for better scientific and medical insights.

Intended Use	A single-use, non-sterile device used as a specimen collector, and for the storage and transport of blood and other biological fluids for analytical and diagnostic analyses.
Product	The intended user is a healthcare professional or a layperson. The blood sample is typically taken through a prick in the fingertip using a lancet. The VAMS [®] sampling tip (contained within the Mitra Device) is gently applied to the drop of blood on the fingertip until the entire sampling tip turns red indicating a volumetric sample (10, 20, or 30 μ L depending on the device being used) has been collected. This process is repeated for each sampling tip in the device. Once all sampling tips (one (1), two (2), three (3), or four (4) depending on the device being used) have been filled, the device outer housing is clicked closed to secure the sample(s).
Storage Instructions	<ul style="list-style-type: none"> • Device Storage: Up to 30°C. For a shorter time (e.g. during transport) up to 45°C. • Sample Storage: The guidelines for sample storage are analyte dependent and will need to be determined by device end-user.
Compliance Certification	Mitra [®] devices are CE-IVD (IVDR) devices intended as specimen collectors and for the storage and transport of dried blood. They are available as registered IVD Devices in the European Union and United Kingdom, Australia, Brazil, China, and Canada, as well as multiple Health Ministries worldwide. In the USA, Mitra devices are supplied as a research use only (RUO) product to assist in method development, other research-related and non-diagnostic activities. Mitra has not been validated for use with any diagnostic testing.
Precautions	<ul style="list-style-type: none"> • Single use only for a single individual. • Do not use after expiration date. • For external use only. • Devices should be transported/mailed to the analytical laboratory, and appropriate documentation maintained according to local regulations and the analytical laboratory procedures and policies. • Do not use if device packaging has been opened or damaged. • Labs must validate use of product for their specific assay. • Observe universal biological risk precautions. • All used materials with blood residues must be handled and disposed of safely in accordance with local regulations. • Any serious incident that has occurred in relation to the medical device should be reported to Trajan Scientific and Medical and the competent authority of the Member State in which the user and/or patient is established.

Technical data	
Product features	
Sample type	Dried Whole Blood
Formats/Configurations	Mitra Device Alone, Mitra Device in Specimen Bag with Desiccant, Mitra Microsample Collection Kit
Sample volume	10, 20, or 30 μ L
Volumetric precision (%RSD)	< 5%
Number of samples collected	Up to 4 (depending on Device used)
Substrate	Polymer
Shelf Life	18 months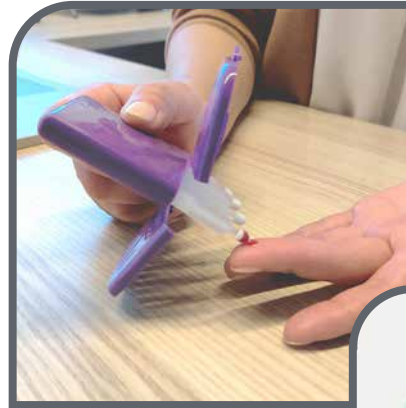

Mitra[®] Device

Benefits of Microsampling

VAMS[®] – the absorbent microsampling technology featured in all Mitra devices – revolutionizes specimen collection. It combines the best of wet and dried blood sampling in state-of-the-art collection devices that are easy to implement at home, in the lab, or in the field.

Collection	
Less-invasive draw technique	✓
Quantitative volume collected	✓
Easy collection outside of the facility	✓
Single-use and tamper-resistant	✓
Transportation/storage	
Native barcode specimen tracking	✓
Ship by regular post as a non-biohazard	✓
No cold-chain shipping required	✓
Specimen is stable when stored at room temperature or in freezer	✓
Laboratory processing	
Compatible with standard liquid handling instrumentation	✓
Compatible with automation	✓
Convenient offline, benchtop processing	✓

Specifications are subject to change without notice.

Neoteryx[®], Mitra[®] and VAMS[®] are registered trademarks owned by Trajan Scientific Australia Pty Ltd.

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Neoteryx[®], Mitra[®] et VAMS[®] sont des marques déposées appartenant à Trajan Scientific Australia Pty Ltd.
Neoteryx[®], Mitra[®] e VAMS[®] sono marchi registrati di proprietà di Trajan Scientific Australia Pty Ltd.
Neoteryx[®], Mitra[®] e VAMS[®] são marcas registadas propriedade da Trajan Scientific Australia Pty Ltd.