Harpera™ microbiopsy tool (IUO)

Intended use


The Harpera™ microbiopsy tool for investigational use only (IUO) is intended to enable the collection of a specimen from the cutaneous skin surface by a healthcare professional for clinical studies.

Product description

The Harpera microbiopsy tool is a single-use manual microbiopsy punch, a hand-held disposable tool intended to achieve a controlled skin puncture in order to obtain a specimen. The tool is supplied individually packaged and has been subjected to gamma irradiation.

Instructions for use

1. Remove tool from packaging, remove protective red cap and remove safety insert.
2. Pull the plunger until a soft click is heard.
The tool is now set.
3. Place the tool vertically against the subject's skin at target site.
4. Press firmly against the subject's skin.
5. Activate the release lever to puncture the skin.
6. Pull the tool with specimen off the skin surface to collect the specimen.

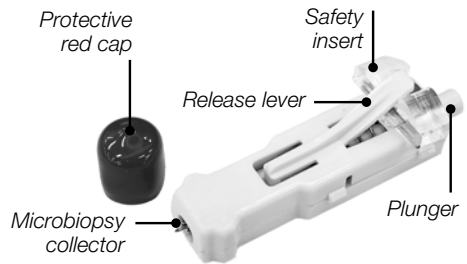Warnings and precautions

- The product has completed a gamma irradiation process for sterilization, but the sterility of the product has not been validated.
- Harpera is a single use only tool.
- Reuse of this tool presents a risk of infection to the patient.
- Handle with care so as not to injure yourself by touching the sharp tips/edges of the microbiopsy collector.
- If the tip of the microbiopsy collector comes in contact with a foreign object, replace the tool with a new one.
- Do not use the tool if any damage is observed to the packaging or the tool itself.
- In the course of the clinical investigation, if an incident occurs with the tool, please contact neo.support@trajanscimed.com as soon as possible.

Storage and handling

In its sterile packaging, the tool can be stored at room temperature. After use, dispose of the tool properly in the biohazard waste. Dispose of the microbiopsy collector as sharp medical waste to prevent infection as per local guidance.

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**Trajan Scientific Australia Pty Ltd**