


instructions for use

Mitra[®] 96-Autorack[™]

Follow these steps to correctly collect precise microsamples.
Questions? neo.support@trajanscimed.com


1

Label samplers and/or 96-Autorack according to your needs/protocol.

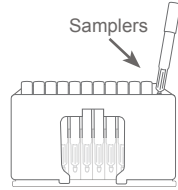

2


Remove 96-Autorack from the collection plate it is resting in by lifting the 96-Autorack up. Set the 96-Autorack down next to the collection plate.

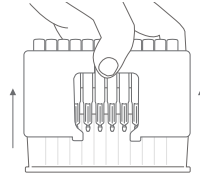

3

Remove a sampler from the 96-Autorack and touch sampling tip to surface of blood sample.

Watch sampling tip go FULLY red and then count 2 seconds. SLOWLY and SMOOTHLY remove sampling tip from blood.

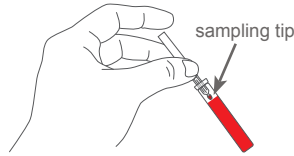


IMPORTANT!

- Do NOT submerge the entire sampling tip in blood
- Tip should always point downward towards floor as illustrated above
- Do NOT drip blood onto the sampling tip

4

Return sampler to 96-Autorack. Repeat step 3 for remaining samplers. Do not touch side walls of 96-Autorack with sampling tip upon entry.

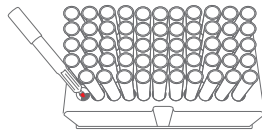


5

Dry sampling tips in rack under ambient conditions (~ 3 hours).


note

The collection plate should ALWAYS be removed from the 96-Autorack during the drying process to allow air to freely circulate around microsamplers. Wait a minimum of 3 hours to ship to ensure dryness of sampling tips.

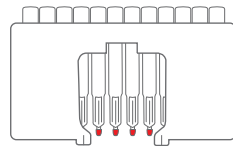

6

Dried samplers can be returned to the collection plate.

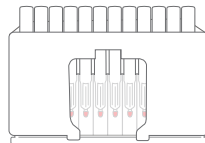

Intended Use

A single-use, non sterile device intended for the collection of specified volumes of blood specimens from a tube in a laboratory setting.

Intended User

This product is indicated for use by healthcare or laboratory professionals to collect blood from a tube for analysis in a laboratory setting.

Disclaimer

The Mitra Microsampling Device is not specific to any clinical test nor does it provide a clinical diagnostic outcome of any nature. Clinical diagnostic laboratories, using the Mitra device for specimen collection, must validate tests according to their organizational needs and comply with relevant regulations.



Trajan Scientific Australia Pty Ltd (Sponsor)
7 Argent Place, Ringwood
Victoria 3134, Australia
Tel: +61 (0) 3 9874 8577



Advena Ltd.
Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta